



PRODUCT INFORMATION

Product Name:
PROCEEDx Respiratory Syncytial Virus Positive Sample

Catalogue Number:
VP-07-01

Product Description:
This product contains inactivated Human respiratory syncytial virus (HRSV) in storage buffer.

SPECIFICATIONS

Strain:
Human respiratory syncytial virus, Long strain. ICTV 01.048.2.01.001

Storage:
Store this preparation at 2 °C to 8 °C.

Shelf Life:
72 weeks from manufacturing date.

Volume:
1 mL

USE

Instructions For Use:
Use PROCEEDx product as per sample handling instructions specified by device or instrument manufacturer.

Intended Use:
The product is to be used **for immunoassay and/or nucleic acid test** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

ASSISTANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

HAZARDS

We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.



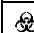
SPECIAL HANDLING

No special handling required for this product.

QUALITY CONTROL INFORMATION

Test Kit	Target	Result
Binax NOW RSV card Catalogue # 430-022	Respiratory Syncytial Virus	RSV Positive
Xpect RSV Catalogue # R24601	Respiratory Syncytial Virus	RSV Positive
Quick Vue RSV Catalogue # 20193	Respiratory Syncytial Virus	RSV Positive
Binax NOW Flu A & B Catalogue # 416-000	Influenza A & B	Influenza A Negative Influenza B Negative
SAS Rotavirus Catalogue # 069020	Rotavirus	Rotavirus Negative
Quick Vue Chlamydia Catalogue # 067020	Chlamydia	Chlamydia Negative

qPCR Test Kit	Analyte	Result
Xpert FLU/RSV Cepheid	Influenza A	Not Detected
Xpert FLU/RSV Cepheid	Influenza B	Not Detected
Xpert FLU/RSV Cepheid	Respiratory Syncytial Virus	Detected

REF	Catalog Number
LOT	Lot Number
	Temperature Limitation
	Expiration Date
	Biological Risk

MICROBIX

