



TECHNICAL INFORMATION SHEET

PRODUCT INFORMATION

Product Name:

PROCEEDx Influenza A and Adenovirus Positive Sample

Catalogue Number:

VP-13-04

Product Description:

This product contains inactivated influenza A and adenovirus in storage buffer.

SPECIFICATIONS

Strains:

Influenza A/H1N1 Subtype

Human adenovirus 2, strain Adenoid 6, ICTV 00.001.0.01.010.00.102.006

Storage:

Store this preparation at 2 °C to 8 °C.

Shelf Life:

72 weeks from manufacturing date.

Volume:

1 mL

USE

Instructions For Use:

Use PROCEEDx product as per sample handling instructions specified by device or instrument manufacturer.

Intended Use:

The product is to be used **for immunoassay** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

ASSISTANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

HAZARDS

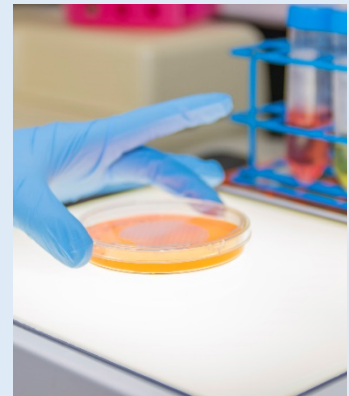
We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.






SPECIAL HANDLING

No special handling required for this product.

QUALITY CONTROL INFORMATION

Test Kit	Target	Result
OSOM Flu A & B Catalogue # 190	Influenza A & B	Influenza A Positive Influenza B Negative
Quidel Flu A & B Catalogue # 20183	Influenza A & B	Influenza A Positive Influenza B Negative
Xpect Flu A & B Catalogue # R24600	Influenza A & B	Influenza A Positive Influenza B Negative
Combi-Strip (Rota and Adeno) Catalogue # C-1004	Rotavirus & adenovirus	Adenovirus Positive Rotavirus Negative
SAS Adeno Catalogue # 067020	Adenovirus	Adenovirus Positive
Xpect Flu A & B Catalogue # R24600	Influenza A & B	Influenza A Positive Influenza B Negative
Xpect RSV Catalogue # R24601	Respiratory Syncytial Virus	RSV Negative



	Catalog Number
	Lot Number
	Temperature Limitation
	Expiration Date
	Biological Risk

