



PRODUCT INFORMATION

Product Name:

PROCEEDx *Trichomonas vaginalis* (TV) Positive Sample

Catalogue Number:

VP-61-01

Product Description:

This product contains inactivated TV protozoan parasite in storage buffer.

SPECIFICATIONS

Strains:

C-1: NIH

Storage:

Store this preparation at 2 °C to 8 °C.

Shelf Life:

24 weeks from manufacturing date.

Volume:

1 mL

USE

Instructions For Use:

Use PROCEEDx product as per sample handling instructions specified by device or instrument manufacturer.

Intended Use:

The product is to be used **for nucleic acid** and/or **immunodiagnostic test** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

ASSISTANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

HAZARDS

We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

SPECIAL HANDLING

It is recommended that the product is tested according to kit Manufacturer's instructions for testing of clinical samples. Before use, the product should be thoroughly mixed by vortexing for approximately 30 seconds. In protocols for the assessment of samples collected by swab procedures, the amount of liquid should be pipetted into the sample transport tube or a cartridge as per test manufacturers' instructions.






QUALITY CONTROL INFORMATION

qPCR Test Kit	Target	Result
GeneXpert TV Test	<i>Trichomonas vaginalis</i>	<i>Trichomonas vaginalis</i> Positive
	Human DNA	Human DNA Positive

Test Kit	Target	Result
OSOM TV Test	<i>Trichomonas vaginalis</i>	<i>Trichomonas vaginalis</i> Positive

EXPECTED RESULTS

Testing of the product in PCR-based and/or immunodiagnostic assays for TV should result in a positive finding for TV, including such assays where sample adequacy is confirmed for the determination of a positive result.

 REF	Catalog Number
 LOT	Lot Number
	Temperature Limitation
	Expiration Date
	Biological Risk

